

Bonvadis®

INSTRUCTIONS FOR USE

Please thoroughly read and understand the instructions for use before using this device.

DEVICE DESCRIPTION

Bonvadis® is a non-sterile water based, preserved, semi-viscous formulation, for prescription use. Bonvadis® contains methyl and propyl parabens which act as preservatives to inhibit the growth of microorganisms within the product before opening and between uses.

Bonvadis® is for multiple use and supplied in a 15 g aluminum tube.

The formulation is to maintain a moist wound environment, the moist wound environment being conducive for wound healing.

INGREDIENTS

Purified water, liquid petrolatum, white petrolatum, propylene glycol, cetyl stearyl alcohol, Tween 60, Span 60, Centella asiatica extract, Mexican mint extract, methyl paraben, and propyl paraben.

INDICATIONS FOR USE

For Prescription Use (Rx)

Under the direction of a healthcare professional, the Rx product is indicated for the management of partial thickness wounds, closed post-surgical wounds, and 1st and superficial 2nd degree burns.

DIRECTIONS FOR USE

- Bonvadis® is for adults topical use only as directed by a Healthcare Professional.
- Clean hands with mild soap or suitable cleanser before use.
- Pierce the head of aluminum tube by using screw top.
- Gently cleanse the wound by a suitable wound cleanser and apply a generous layer of Bonvadis® on the surface of the wound.
- Cover Bonvadis® with an appropriate secondary dressing, such as gauze.
- After use, close the tube cap.
- Reapply as necessary once or twice daily (suggest 0.5 g per day) or as directed until the wound or lesion has healed fully.

CONTRAINDICATIONS

- Do not use Bonvadis® if you have a known sensitivity or allergies to any Bonvadis® ingredients.
- Do not use for the treatment of third degree burns or to control heavy bleeding.

WARNINGS AND CAUTIONS

- Keep out of the reach of children.
- Bonvadis® should not be ingested or taken internally. If ingested or taken internally, contact a doctor immediately.
- The product should be discarded after use of 60 days. Preservative effectiveness testing has not been conducted on the product to support the use for more than 60 days.
- Bonvadis® does not contain a sunscreen and should not be used prior to extended exposure to the sun.
- If the observation of signs of infection on the wound, appropriate treatment should be initiated under a healthcare professional's direction.
- Dispose with domestic waste after sealing.

SHELF LIFE AND STORAGE










- Bonvadis® cannot be used after the expiry date. The expiry date is stated on the package.
- Bonvadis® should be stored away from sunlight. Storage and transport at 15°C to 30°C.
- Operate at room temperature (15°C to 30°C).
- Do not store in freezer.

HOW SUPPLIED


Bonvadis® is supplied in tubes in the following sizes:

- 15 g (0.03 lb)

SYMBOL DEFINITION

 Non-sterile	 Use by Date	 Caution, Consult Accompanying Documents
 Batch Number	 Consult Instructions for Use	 Do Not Use if Package is Damaged
 Temperature Limit	 Rx only Prescription Devices	 Manufacturer

ONESS
Oneness Biotech Co., Ltd.

 **Oneness Biotech Co., Ltd.**
No. 8, Tangchang Rd., Nanchou Township, Pingtung County 926, Taiwan
TEL: +886-8-864-1088